

Model 610 Washer/High Level Disinfecter Features

Product Description

- A self contained, process-controlled, Washer/High Level Disinfecter.
- Includes: Process Data Recorder for JCAHO documentation, 1 drain hose, 4 compartment dividers within 3 multi-purpose trays and 1 primary tray, 10 mesh bags, 1 gallon of Clean 505 Cleaning Fluid with MSDS, 1 – 24 ounce of CaviCide with MSDS, 1 operation/service manual, and 5 cotton tip applicators

Load Processing

- A 25 gallon, circular, stainless steel tank processes approximately 2.5 cubic feet of equipment in each load.
- Trays and compartment dividers partition the tank to customize each load. Mesh bags speed loading and unloading without excess handling of equipment.

Cycle Description

- A WASH ONLY cycle consisting of a 5 minute wash and two full immersion rinses and drains. Approximately 30 minute cycle time.
- A WASH/PASTEURIZE cycle consisting of a full WASH cycle followed by a 30 minute timed bath with 160°F (71°C) minimum temperature water. Approximately 60 minute cycle time.
- A SYSTEM CLEAN cycle consisting of manual cleaning by the user followed by full immersion rinses and drains.

Cleaning Action

- Horizontal agitation occurs 60 times per minute for vigorous scrubbing and optimum bubble removal during fill, wash, and rinse cycles. Agitation during each drain cycle expels water from equipment to speed drying.

- An external spray nozzle is available for pre-rinsing medical devices prior to start of any cycle or for cleaning inside the tank.

Controls

- Cenorin's process control software automation manages all aspects of the cleaning and high level disinfection processes. Audible signal and visual notification on screen when the cycle is complete.
- Display allows monitoring of cycle status, remaining time, and tank temperature.
- The Process Data Recorder provides a printed record of all cycles with time, date; cycle selected, and tank temperatures every 5 minutes.
- Time and temperature are monitored by the microprocessor to guarantee that pasteurization is achieved (30 minutes at 160°F (71°C) ± 2°F (1°C)).

Warranty

- A 2 year warranty covering 100% of parts and labor is standard.
- An additional five years of warranty (parts only) may be purchased with the Extended Warranty Plan (EWP).

Service

- Warranty service is provided through Medical Equipment Repair Association (MERA), with over 85 offices nationwide, or through in-house biomedical staff.
- Toll free phone number for technical support for the life of the equipment: 800-426-1042.

#224861  _{US}

High Level Disinfection Systems for Healthcare

Cenorin, LLC • 6324 South 199th Place • Suite 107 • Kent, WA 98032
Phone: 253-395-2400 • Fax: 253-395-2650 • Toll Free: 1-800-426-1042
FDA Registered • CSA Certified • ISO 13485:2003