

Pasteurizer Model 520/540 Preventive Maintenance Schedule

Weekly (minimum)

- Tanks: Use a powdered cleanser for stainless steel (*Cameo* or *Bar Keeper's Friend*) to thoroughly clean all stainless steel parts. Rinse completely with water.
- Drain screens: Remove all debris and clean with a tooth brush. Remove screen if necessary.
- Water inlet and air dome port: Clean with a tooth brush.
- Exterior surfaces: Clean with liquid cleanser suitable for painted surfaces.
- Interior surfaces: Disinfect with anti-microbial disinfectant (*CaviCide*) and wipe down all stainless steel surfaces, including tanks, lids and ports. Use a large cotton tipped applicator saturated with disinfectant for the water inlet port and the air dome port. Allow CaviCide to dry for 10 minutes before rinsing and drying.

Note: To be effective as a disinfectant, all anti-microbial agents need to remain in contact with the surface for a length of time. Refer to the package label for instructions on proper use if not using CaviCide.

Biannually

- Check temperature calibration: Compare actual and displayed tank temperature. Use a thermometer to measure water temperature in the tank. Actual temperature should be +/- 2-3°F (1°C) of the displayed temperature. Refer to the Service Manual – Temperature Display Calibration for instructions.
- Change the drive block and stop ring: Order part number 082380-01 or 082380-04 PM kit, Model 520/540 (see below).
- Replace the water seal: Included in PM kit. Instructions in the Service Manual – Water Seal Replacement.

Recommend *CaviCide* disinfectant/cleaner. Listed below are maintenance parts.

Part #	Description
020590-00	<i>CaviCide</i> disinfectant/cleaner – 24 ounces
020590-01	<i>CaviCide</i> disinfectant/cleaner – case of 12
020591-00	Cotton tipped applicator – box of 100
082380-01	PM Kit, Model 520/540 - manufactured between 1986 and July 2002
082380-04	PM Kit, Model 520/540 - manufactured in or after August 2002
082380-00	PM Kit Spanner Wrench – for systems manufactured in or after August 2002

High Level Disinfection Systems for Healthcare

Cenorin, LLC • 6324 South 199th Place • Suite 107 • Kent, WA 98032
Phone: 253-395-2400 • Fax: 253-395-2650 • Toll Free: 1-800-426-1042