

Model 540 Washer/Pasteurizer Specifications

Product Description

- A self contained, microprocessor controlled, High Level Disinfection washer/pasteurizer.
- Includes: Process Data Recorder for JCAHO documentation, parts handling basket, 3 layer dividers, 4 compartment dividers, 10 mesh bags, 4 pound tub of CLEAN 505 detergent, 1 operation manual, and 1 service manual.
- Display allows monitoring of cycle status, remaining time, and tank temperature.
- The Process Data Recorder provides a printed record of all cycles with time, date, cycle selected, and tank temperatures every 5 minutes.
- Pasteurization water temperature is maintained at 160-170°F ± 2°F (75°C) by microprocessor controlled heat bands.
- Time and temperature are monitored by the microprocessor to guarantee that pasteurization is achieved. (30 minutes at 160°F or more.)

Load Processing

- A 25 gallon, circular, stainless steel basket processes approximately 2.5 cubic feet of equipment in each load.
- Layer and compartment dividers partition the basket to customize each load. Mesh bags speed loading and unloading without excess handling of equipment.

Cycle Description

- A WASH ONLY cycle consisting of a 5 minute wash and two full immersion rinses and drains. Approximately 30 minutes cycle time.
- A WASH/PASTEURIZE cycle consisting of a full WASH cycle followed by a 30 minute timed bath with 160°F (71°C) minimum temperature water. Approximately 60 minute cycle time.

Cleaning Action

- Horizontal agitation 60 times per minute for vigorous scrubbing and optimum bubble removal during fill, wash, and rinse cycles. Agitation during each drain cycle shakes water from equipment to speed drying.

Controls


- Microprocessor controls for all cycle timing and temperature processes. Audible signal when the cycle is complete.

Warranty

- A 2 year warranty covering 100% of parts and labor is standard.
- An additional five years of warranty (parts only) may be purchased with the Extended Warranty Plan (EWP).

Service

- Warranty service is provided through Medical Equipment Repair Association (MERA), with over 85 offices nationwide, or through in-house biomedical staff.
- Toll free phone number for technical support for the life of the equipment. 800-426-1042.

#224861 

High Level Disinfection Systems for Healthcare

Cenorin, LLC • 6324 South 199th Place • Suite 107 • Kent, WA 98032
Phone: 253-395-2400 • Fax: 253-395-2650 • Toll Free: 1-800-426-1042